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gb AVIZOR **PRESERVATIVE FREE**
lacrifresh
moisture 0,10%
Unidose

Usage:
Lacrifresh Moisture Unidose soothes symptoms of ocular irritations, dryness and itredness. Its special formula contains Sodium Hyaluronate, a natural substance present in the eye, that forms a uniform and protective shield with a great water absorption capacity and resistant to evaporation. The Lacrifresh Moisture Unidose protects your eyes with an excellent hydration thus improving visual comfort and lens wear. They are especially recommended for persons in environments where ocular irritations and dryness increase.

Instructions for Use:

- Take an Lacrifresh Moisture Unidose vial and open it by turning the cap
- Direct use: Apply a few drops into each eye and blink several times.
- Use on contact lenses: Apply a few drops into the internal part of each lens before inserting.
- Use Lacrifresh Moisture Unidose as many times as needed.

Last revision date: 2009/11

Precautions

- Once the vial is open, use immediately. This product does not contain preservatives and after use it must therefore be discarded to prevent a microbial contamination.
- Do not use if the blister is damaged.
- Do not ingest the solution and keep out of children's reach.
- Do not use the product after the expiry date on the box
- Store at room temperature.
- Do not use the product in case you are allergic to any of its components.
- In case of ocular irritation or adverse reaction please consult a specialist.
- Before using any ophthalmic medicine, inform your doctor you are a contact lens user.

Composition:
Sodium Hyaluronate 0,10 %, in a isotonic, buffered and aqueous solution.

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